Berkeley Life


Nitric Oxide Test Strips

Results you can see. Precision you can rely on.



Berkeley Life's patented, non-invasive Nitric Oxide Test Strips measure your patients' salivary nitrite levels in just 10 seconds. Our strips provide an easy and accurate way for patients to track their body's Nitric Oxide levels from home.

Get accurate results in 10 seconds.

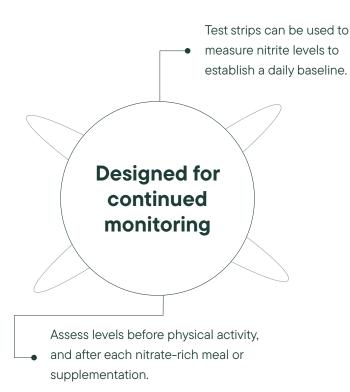

The test strips can be used to measure nitrite levels and establish a daily baseline. Encourage your patients to check their nitrite levels before physical activity and after each nitrate-rich meal or supplementation.

TEST Nitric Oxide levels using a test strip placing the 'saliva here' side straight down on the tongue.

ASSESS Nitric Oxide levels against the color chart by folding the strip in half and pressing the two pads together to reveal your current level.

TAKE two capsules of the Berkely Life Nitric Oxide Foundation and retest in 90 minutes.

Very Low Low Threshold Target High

*These statements have not been evaluated by the Food and Drug Administration. This product is not intended to diagnose, treat, cure or prevent any disease.